

5 Depth Adjustable

Lancing Device With Ejector

Instructions for the Lancing Device


This *Lancing Device With Ejector* is used to obtain capillary whole blood samples needed for blood glucose monitoring or for other tests requiring one or two drops of blood. The *Adjustable Tip* on the *Lancing Device* allows you to choose the best depth of skin penetration. The ejector is more safe professional system for the lancet disposal. To obtain a drop of blood, first wash your hands with warm, soapy water. Dry your hands thoroughly before using the *Lancing Device*.

Directions

1. Twist off the *Lancing Device Cover*.
2. Insert a new *Lancet* firmly into the *Lancet Carrier*.
3. Hold the *Lancet Needle Cover* and gently twist it off of the *Lancet*.
4. Replace the *Lancing Device Cover*.
5. The *Adjustable Tip* offers 5 levels of skin penetration. To select the desired depth, twist the *Adjustable Tip* in either direction until the numbers line up with the *Arrow*.
To select the best depth: 1-2 for soft or thin skin
3 for average skin
4-5 for thick or calloused skin
6. Hold the *Lancing Device Cover* in one hand. Using the other hand, slowly pull the *Sliding Barrel* away from the *Lancing Device Cover*. You will hear a click, indicating that the *Lancet Carrier* is locked into position. Release the *Sliding Barrel* to return it to its original position.

Taking Care of Your Lancing Device With Ejector

Remove the *Lancing Device Cover*. Wash the *Lancing Device Cover* in warm, soapy water. Rinse well and allow it to dry thoroughly. Use alcohol or mild soap and water to wipe the remaining outside of the *Lancing Device*. The *Lancing Device* can be disinfected using 70% rubbing alcohol. (Hospitals should consult their own infection control protocols.)

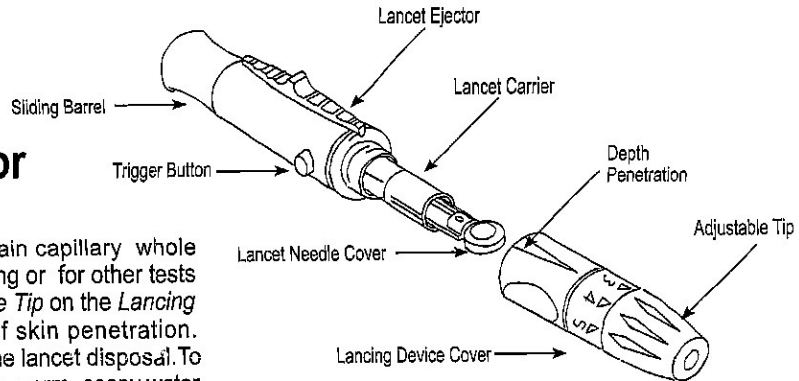

7. Position the end of the *Adjustable Comfort Tip* against the side-tip of the finger. Press the *Trigger Button*, and then lift the *Lancing Device* away from the finger after the puncture is complete.
8. Place the *Lancing Device* aside and wait a few seconds for a blood drop to form. To help form a blood drop, lower the hand to waist level and gently massage the finger from palm to fingertip.
9. To remove the used *Lancet* from the *Lancing Device*, first twist off the *Lancing Device Cover*. While holding the *Sliding Barrel*, leave the *Lancet* in the *Lancet Carrier* and firmly stick the *Lancet Needle* into the *Lancet Needle Cover*.
10. Sliding the *Lancet Ejector* lever in a forward motion and disposing the used lancet in an approved container. Press the *Trigger Button* to release the *Lancet Carrier*. Replace the *Lancing Device Cover*. DO NOT re-use the *Lancet*. If the *Lancing Device* is to be used by another person, provide a new *Lancing Device Cover* and a new *Lancet*.

Limited Warranty

The *Lancing Device* will function effectively for a period of six (6) months from the date of purchase. A malfunctioning device should be returned to your supplier for replacement.